



[Login](#) | [Register](#) | [Order Status](#) | [Contact Us](#) | 1.800.558.9595

[Home](#) [Arts & Crafts](#) [Education](#) [Farm Supplies / Ag Science](#) [Health Training](#) [Lab Sampling / Whirl-Pak®](#) [Senior Activities](#)

Search in all Catalogs



[Quick Order Form](#)

[My Basket](#)



[Sign-up for Email Updates](#)

เลือกภาษา ▼

This is a translatable version of the instruction manual. To translate, Select Language from the drop down menu in the upper right corner. Please see [Google Translate Disclaimer](#).



[Print this manual](#)

## Suture and Stapling Practice Arm LF01028U Instruction Manual



**Life/form® Products by Nasco**



**Nasco**  
Dedicated to Delivery


901 Janesville Avenue, P.O. Box 901 • Fort Atkinson, Wisconsin 53538-0901  
1.800.558.9595 • [eNasco.com](#) • E-mail: [lifeform@eNasco.com](mailto:lifeform@eNasco.com)  
COPYRIGHT © NASCO 1982 PRINTED IN U.S.A. NP 30-82/RV 7-15

### About the Simulator

This practice and demonstration suture arm should be considered disposable, but with proper care and treatment, it will serve its purpose for a long period of time.

The three "wounds" provided should be considered only samples of what can be done. With a sharp scalpel, additional "wounds" can be cut at any desired depth or location. Each can be repeatedly sutured until the skin at any one site becomes so heavily perforated that it will no longer support more suturing.

The "wounds" can be bandaged with most standard adhesive bandages. Residual adhesive can be removed with Glue-Off (LF01023U).

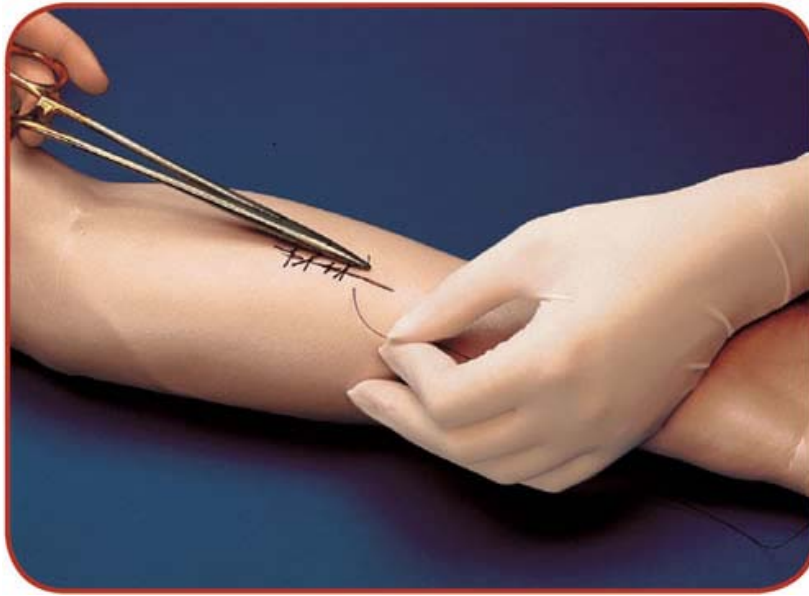

### Care of Simulator

Do not use providone-iodine prep solutions. These solutions will indelibly stain the arm.

Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with a soft cloth or paper towels.

When not in continuous use, it is recommended to store the simulator in the storage box furnished, or cover it with a cloth or sheet of plastic.

### Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball-point pens will also make indelible stains.

### Supplies/Replacement Parts for Suture Practice Arm

LF01011U Suture Tool Kit  
LF09919U Nasco Cleaner

**Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.**

#### Shopping Help

Bilingual Products  
How to Place an Order  
Search Help  
Program Supplies  
Register or Login  
Summer Camp  
Prop 65 Q & A's

#### Distributors-Dealers-Vendors

Got a New Product for Us?  
Purchase Order Terms & Conditions  
Sales Directors and Buyers  
Second Source Form  
Supplier Guidelines  
Vendor Forms

#### Safe & Secure Shopping

Check Order Status  
Coupon Exceptions  
Frequently Asked Questions  
International Orders  
Privacy Policy  
Shipping & Delivery  
Translation Disclaimer





#### Company Information

About Us  
Customer Service  
Catalog Outlet Stores  
Employment Opportunities  
Request a Catalog  
Site Directory  
Sustainability

**Life/form** **NASCOGuard**

#### 2016 Updates

Education Purchasing  
Department Essentials

 Find us on Facebook  
 Follow us on Twitter  
 eNasco Blog  
 YouTube

Prices listed are U.S. Domestic prices only and apply to orders shipped within the United States. Orders from outside the United States may be charged additional distributor, customs, and shipping charges.

Customer Service 1.800.558.9595 Copyright © 2016 Nasco

